RI.MOS.
Medical Products

RI.MOS. Srl Via Manuzio, 15 - 41037 Mirandola (MO) - ITALY - info@rimos.com

**LESIONI
COMPLESSE**

Hyperoil®

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**COMPLEX
SKIN LESIONS**

Hyperoil®

Class IIb medical product
For use on the skin only

Use

Hyperoil® is an interactive dressing that assists skin tissue regeneration and repair in acute and chronic lesions which usually heal by second intention. The product is recommended for complex skin lesions of various nature: *post-surgical and traumatic wounds, burns, ulcers, pressure ulcers, radiodermatitis, vascular and diabetic lesions, lesions with dermal and/or of bone tissue exposure, management of skin grafts.*

Barrier effect helps maintaining proper tissue hydration at the site of the lesion reducing eschar formation, keeps bacterial growth unchanged and reduces cheloid formation.

Applications

The product can be used both in hospitals and at home, under the guidance of health workers or family members and care assistants suitably trained in the use of the product.

CAUTION: complex skin lesions always require the supervision of a doctor or of authorized medical personnel.

Method of application

OILY AND GEL FORMULATION

Clean the wound with water or physiological saline, apply a very small quantity of Hyperoil®. Cover the wound with a breathable patch or with TNT sterile gauze so that external factors do not cause infections. Repeat the medication according to indications of the doctor, until complete restoration.

IMPREGNATED GAUZES

If necessary, clean the wound with water or physiological saline, apply a gauze Hyperoil® on the wound. If necessary, cover the wound with a breathable patch or with TNT sterile gauze so that external factors do not cause infections. Repeat the medication according to indications of the doctor, until complete restoration. The gauze is disposable and must be changed each time the medication is applied.

Limits to use

The product is not effective on necrotic tissue if tissue hypoxia is less than 30 mmHg di O₂.

Contraindications

Do NOT use the product on the treated area at the same time as other ointment or disinfectants of any type.

Improper use of the products can cause a delay or absence of tissue repair and regeneration, impeding the healing process.

In the event of hypersensitivity to the active ingredients or excipients in the product, treatment must be discontinued and a suitable alternative therapy implemented.

The application of the products, in any of its formulations, may lead to the onset of a temporary sensation of burning, itching or reddening around the damaged area, this favours a gradual revascularisation at the site of the lesion.

Warnings

- In most cases, it allows for avoiding the use of local antibiotic treatment and/or the shortening of systemic antibiotic treatment.
- Avoid the use of disinfectants as they can have a negative effect on the tissue regeneration and repair process.
- Keep the product air-tight and in the dark. Expose it to the air only for the amount of time necessary and seal the container properly after use.
- Store the Oily Formulation at temperature between +20°C and +35°C. At temperatures below 17°C, the product tends to solidify, but as soon as it is applied to the skin, it immediately liquefies, without altering the active ingredients.
- Store the Gel Formulation and the Impregnated Gauzes at temperature between +10°C and +25°C.
- The product is made of natural vegetable substances and is NOT dangerous for the health or the environment.
- Allergies that have not previously been noted can develop.
- Do not ingest.
- The product has not been clinically tested on 2nd and 3rd degree burns.
- Keep out of the reach of children under three years of age.
- The oily formulation must be shaken well before use.
- Due to the presence of Neem Oil, this product is not suitable for those intending to conceive.
- Do not use if the packaging is open or damaged.
- The expiry date indicated on the package refers to the product in intact packaging, correctly preserved.

PETROLATUM AND PARABEN FREE

Oily formulation INCI:

Melia Azadirachta Seed Oil, Hypericum Perforatum Flower Extract.

Gel Formulation INCI:

Water; Propylene Glycol; Melia Azadirachta Seed Oil; Cyclodextrin; Aroma; Triethanolamine; Phenoxyethanol; Carbomer; Hypericum Perforatum Flower Extract; Benzoic Acid; Dehydroacetic Acid; Sodium Dehydroacetate; Ethylhexylglycerin.

Impregnated Gauzes INCI:

Cotton gauze; Hyperoil® Gel for gauzes (Water; Propylene Glycol; Melia Azadirachta Seed Oil; Cidodextrin; Triethanolamine; Phenoxyethanol; Carbomer; Hypericum Perforatum Flower Extract; Benzoic Acid; Sodium Dehydroacetate; Ethylhexylglycerin).

Warning: consult instructions for use

www.rimos.com

CE 0051

Please dispose of carefully after use

RI.MOS.
Medical Products

RI.MOS. srl Via Manuzio, 15 - 41037 Mirandola (MO) - ITALY - info@rimos.com

**COMPLEX
SKIN LESIONS**

Hyperoil®

Class IIb medical product
For use on the skin only

PAO (period after opening) 100ML GLASS BOTTLE 24 months

PAO (period after opening) 75ML GEL TUBE 12 months

PAO (period after opening) 5ML VIAL 2 months

KEEP AWAY FROM DIRECT SUNLIGHT	STORE IN A COOL DRY PLACE	STORE AT TEMPERATURE BETWEEN +XX°C AND +YY°C	DO NOT USE IF PACKAGE IS DAMAGED	RECYCLED AND/OR RECYCLABLE MATERIAL
LOT BATCH CODE	REF PRODUCT CODE	ESTIMATED QUANTITY	MANUFACTURER	MEDICAL DEVICE
DATE BY WHICH THE PRODUCT CAN BE USED	CAREFULLY READ THE PRECAUTIONS	CAREFULLY READ THE INSTRUCTIONS FOR USE	NON STERILE	AFTER USE DO NOT DISPOSE OF INTO THE ENVIRONMENT
				AFTER OPENING DO NOT USE BEYOND THE MONTHS SHOWN

COD.INT. ISTCOMP

REV. 0 - MARZO 2024

www.rimos.com

CE 0051

Warning: consult instructions for use

Please dispose of carefully after use