### Destinazione d'uso

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**SIMPLE  
SKIN LESIONS**

**Hyperoil®**

**Class IIb medical product  
For use on the skin only**

**Use**

Hyperoil® is an interactive dressing that assists skin tissue regeneration and repair in acute and chronic lesions which usually heal by second intention. The product is recommended for superficial or deep skin lesions of various nature: *burns, traumatic wounds, eczema, urticaria, erythema, scrapes, haemorrhoids, fistula, rhagades.*

Barrier effect helps maintaining proper tissue hydration at the site of the lesion reducing eschar formation, keeps bacterial growth unchanged and reduces cheloid formation.

**Method of application**

Clean the wound with water or physiological saline, apply a very small quantity of Hyperoil® and massage it until it is totally absorbed. If necessary, cover the wound with a breathable patch or with TNT sterile gauze so that external factors do not cause infections. Repeat the medication daily, until complete restoration.

**Limits to use**

The product is not effective on necrotic tissue if tissue hypoxia is less than 30 mmHg di O<sub>2</sub>.

**Contraindications**

Do NOT use the product on the treated area at the same time as other ointment or disinfectants of any type. Improper use of the products can cause a delay or absence of tissue repair and regeneration, impeding the healing process. In the event of hypersensitivity to the active ingredients or excipients in the product, treatment must be discontinued and a suitable alternative therapy implemented. The application of the products may lead to the onset of a temporary sensation of burning, itching or reddening around the damaged area, this favours a gradual revascularisation at the site of the lesion.

**Warnings**

- In most cases, it allows for avoiding the use of local antibiotic treatment and/or the shortening of systemic antibiotic treatment.
- Avoid the use of disinfectants as they can have a negative effect on the tissue regeneration and repair process.
- Keep the product air-tight and in the dark. Expose it to the air only for the amount of time necessary and seal the container properly after use.
- Store at temperature between +10°C and +25°C.
- The product is made of natural vegetable substances and is NOT dangerous for the health or the environment.
- Allergies that have not previously been noted can develop.
- Do not ingest.
- The product has not been clinically tested on 2nd and 3rd degree burns.
- Keep out of the reach of children under three years of age.
- Due to the presence of Neem Oil, this product is not suitable for those intending to conceive.
- Do not use if the packaging is open or damaged.
- The expiry date indicated on the package refers to the product in intact packaging, correctly preserved.

*PETROLATUM AND PARABEN FREE*

**INCI:**

Water; Propylene Glycol; Melia Azadirachta Seed Oil; Cyclodextrin; Aroma; Triethanolamine; Phenoxyethanol; Carbomer; Hypericum Perforatum Flower Extract; Benzoic Acid; Dehydroacetic Acid; Sodium Dehydroacetate; Ethylhexylglycerin.

**PAO (period after opening) GEL TUBE ..... 12 months**

**PAO (period after opening) 5ML GEL VIAL ..... 2 months**

KEEP AWAY FROM DIRECT SUNLIGHT	STORE IN A COOL DRY PLACE	STORE AT TEMPERATURE BETWEEN +XX°C AND +YY°C	DO NOT USE IF PACKAGE IS DAMAGED	RECYCLED AND/OR RECYCLABLE MATERIAL
BATCH CODE	PRODUCT CODE	ESTIMATED QUANTITY	MANUFACTURER	MEDICAL DEVICE
DATE BY WHICH THE PRODUCT CAN BE USED	CAREFULLY READ THE PRECAUTIONS	CAREFULLY READ THE INSTRUCTIONS FOR USE	NON STERILE	AFTER USE DO NOT DISPOSE OF INTO THE ENVIRONMENT
				AFTER OPENING DO NOT USE BEYOND THE MONTHS SHOWN

COD.INT. ISTSEMP

REV. 0 - MARZO 2024

[www.rimos.com](http://www.rimos.com)

**Warning: consult instructions for use**

**CE** 0051

**Please dispose of carefully after use**